



MINT FLAVORED



SPRAY

Mint flavored PIP Spray is a refreshing tasting intraoral wetting agent. Its primary purpose is to use in conjunction with Mizzy Pressure Indicator Paste (PIP), although it has other advantages in the operator.

To Eliminate Heavy Ropy Saliva:

1. Spray PIP Spray into patient's mouth.
2. Have patient swish it around and spit it out.
3. In extreme cases wipe palate with tissue or swab and repeat spray.

For Greatly Improved Alginate or Hydrocolloid Impressions (Smooth, Accurate, Bubble-Free):

1. Spray teeth and soft tissues with PIP Spray before taking impression. PIP Spray eliminates saliva bubbles and puts a free-flowing inert barrier over mucin and salivary chemicals that might otherwise react with alginate to produce a fuzzy impression.

To Prevent Mirror Fog

Dentists who work with water spray find that their mirrors both fog up and hold water resulting in poor visibility. Try spraying your mirror with PIP Spray. Because it is such an effective wetting agent, visibility will be greatly increased.