Lustrol

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Lagerung

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Lustrol

Silicone based gloss varnish, for sealing and smoothing of soft denture relinings Molloplast® B and mollosil®

Fig. 1

Fig. 2

Intended use

Silicone gloss varnish for definitive silicone relinings

Preparation

Prior to the application of Lustrol gloss varnish base and functional borders of the relined denture have to be absolutely clean and dry!

Mixing ratio

1 drop of Lustrol: 1 drop of catalyst

Mixing

Mix homogeneously gloss varnish and catalyst according to instructions on a mixing pad or glass slab (Fig. 1).

Mixing time

½ minute

Application

Apply Lustrol with a brush in a thin layer and let it dry for approx. 15-20 minutes (Fig. 2)

Setting time

10 minutes from initiation of mixing

Caution

Avoid contact of catalyst with eyes and skin, this can lead to irritation. In case of accidental contact immediately flush under running water for at least 15 minutes and seek immediately medical care.

Note

- Use enclosed pipettes as closure caps and also for dosing, after removal of the original bottle tops.
- Wear suitable eye protection and gloves!
- Please follow the instructions of the safety data sheet!

Only for dental use by qualified personnel.

Application

At 23 °C/73 °F, 50 ± 5% rel. humidity

Storage

Always keep containers tightly closed.

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GmbH & Co. KG