Modralit® 3K

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Modralit® 3K

Application:

Filled, polyurethane based preparation material.

Modralit® 3K is excellently suitable for the manufacture of precision models for the whole precious metal and non precious metal technique, especially for the dimension stable reproduction of step- and inlay or onlay preparations.

Characteristics:

Modralit® 3K is pourable, easily flowable, draws exactly, is stable in dimension, edge - and abrasion resistant, easily grindable and sawable.

Quantity:

Modralit® 3K is delivered in glass bottles (200 ml base material, 100 ml hardener), as well as 400 g filler material. The package encloses a dosage spoon for the filler, mixing and dosage beaker as well as a mixing spatula.

Information on processing:

Before processing, the glass bottles with the base material and the hardener are shaken. The screw caps are colour marked and therefore cannot be mixed-up. After use the glass bottles have to be closed thoroughly.

- To use the complete processing time first basis material and filler (for a complete tooth arch approx. 10 ml basis liquid and a level spoon filler) have to be filled into the enclosed dosage and mixing beaker and have to be mixed. Afterwards the necessary hardener liquid is added to the mixture and then thoroughly stirred until reaching homogeneity.

Mixing ratio:

Base liquid	Filler	Hardener liquid
10 ml	1 dosage spoon (ca. 14 g)	5 ml

In the case of need the applied quantity can be varied according to the mixing ratio.

- After a mixing time of about 30 sec. let Modralit® 3K flow as a thin stream into the cleaned impression. The flowing phase lasts 2 min. at 20°C (68°F).
- In order to avoid air bubbles, after pouring out, Modralit® 3K can be hardened in a dry pressure pot at 2 - 4 bar. After one hour Modralit® 3K is hardened so far that the tooth arch can easily be trimmed, drilled and grinded.
- Modralit® 3K adapts all known preparation- and distance lacquers and is resistant towards means of insulation.

Important indications:

- Modralit® 3K is available in the primary colour ivory.
- Do not expose Modralit® 3K to unnecessary warmth. Use stream jet for cleaning (not longer than 15 sec.). A thoroughly controlled processing offers you a preparation of first-class quality.
- If using polyether impression resp. duplicating mass a releasing agent (Separator D4305 or Duosep D4306) **has to** be used.
- If using hydrocolloid-materials the area which has to be filled with Modralit® 3K has to be carefully dried with pressure air.
- Can cause allergic reactions at sensitive persons in reference to some materials.
- A disinfecting is possible with all current materials.
- The product is exclusively designed for the field of application described above and only to be used by qualified and instructed persons.
- The indications for this product are based on intensive, application-technique experience and research. Due to ongoing developments the product is subject to alterations.

Possible dangers

Before use observe the Material Safety Data Sheet of Modralit® 3K.

Modralit® 3K Base

Hazard statements

H315 Causes skin irritation.
H319 Causes serious eye irritation.

Precautionary statements

P280 Wear protective gloves/protective clothing/eye protection/face protection.
P264 Wash thoroughly after handling.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P332+P313 If skin irritation occurs: Get medical advice/attention.
P337+P313 If eye irritation persists: Get medical advice/attention.
P302+P352 IF ON SKIN: Wash with plenty of water.

Available in:

- REF** D3926 Modralit® 3K
- REF** D39261 Modralit® 3K, 100 ml base
- REF** D39262 Modralit® 3K, 100 ml hardener
- REF** D39263 Modralit® 3K, 400 g filling material
- REF** D4305 Separator, 50 ml

Lot number / Best before date: The lot number and the best before date are indicated on each Modralit® 3K packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date.

2017-03-13/Rev.2

Drewe Dentamid GmbH

Max-Planck-Str. 31, 59423 Unna / Germany
Tel.: +49 2303 8807-40 Fax: +49 2303 8807-55
www.drewe.com/dentamid dentamid@drewe.de