



Product information

Innodes IMP Impression Disinfection

2500 ml REF 5013K



Characteristics

- Effective in just **1 minute!** (Additional TB effectiveness after 15 min.)
- Contains no aldehyde, alcohol or toxic ingredients – safe to use!
- Pleasant scent
- Licensed as a medical product

Product type

Amine-based ready-for-use solution. Free of aldehydes and has a pleasant scent.

Areas of application

For the disinfection of all dental impression materials.

Application

Insert the impression in question into the ready-for-use disinfection solution and remove it again after the recommended application time. The impression must be fully covered by the solution. Only immerse alginates and hydrocolloids for a short time and let them dry. Then rinse the impression with water. Do not mix with cleaning agents or other disinfectants. Depending on the degree of contamination, the working solution may need to be replaced (RKI recommendation: at least once per working day). The working solution can also be disposed of in the sewage water system.

Application times

Bacteria (incl. MRSA) as per VAH, EN 13727 and EN 14561 (dirty conditions):

Conc. 1 min.

Candida albicans as per VAH, EN 13624 and EN 14562 (dirty conditions):

Conc. 1 min.

Enveloped viruses (e.g. HBV, HCV and HIV) as per RKI/DVV (dirty conditions):

Conc. 1 min.

Chemical-physical data

Density: 1g/cm³

pH-value: 7.2

Available containers

- Plastic bottles

Certificates

Our products have been scientifically examined and tested in accordance with the latest methods. We will be happy to send you copies of our assessments on request.

TbB (M. terrae in a suspension test, clean conditions):

Conc. 15 min.

Noroviruses as per EN 14476, (clean conditions):


Conc. 30 min.

Rotaviruses as per BGA/DVV, (clean conditions):

Conc. 1 min.

Safety guidelines

H 412 Harmful to aquatic life with long lasting effects. Avoid release to the environment. Dispose of the contents/container in accordance with local/regional/national/international regulations.

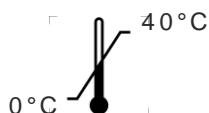

Contact

PRISMAN Pharma International AG

Am Stalden 16

4622 Egerkingen

Switzerland

Tel.: +41 (0)62 206 20 60

Fax: +41 (0)62 206 20 69

www.innocid.com

info@prismanpharma.com

Manufacturer

PRISMAN GmbH

Otto-Hahn-Ring 6-18

64653 Lorsch

Germany